




ExperTemp® Dispensing Gun Instructions

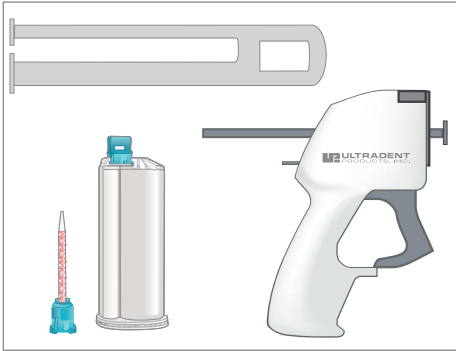


COMPONENT ASSEMBLY

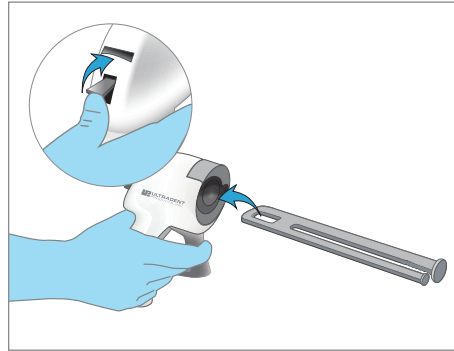


Push up the lever and insert the plunger until it stops.

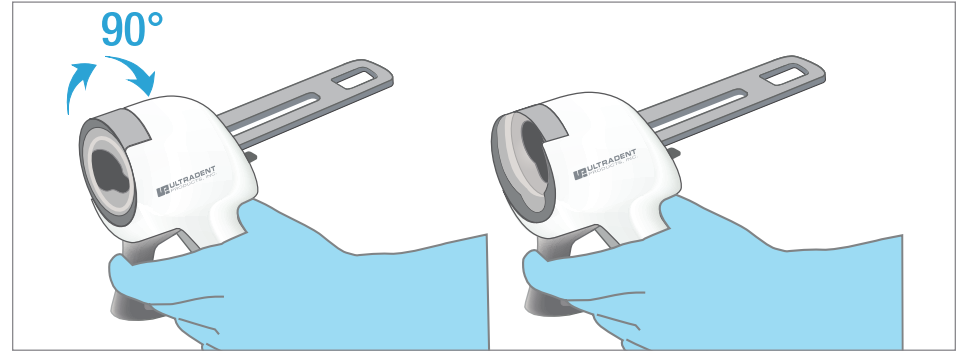


To open dispenser, rotate the ring 90° clockwise until it locks in place.

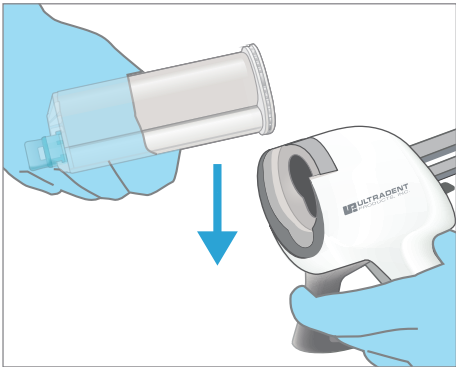


Insert the cartridge into the rotation ring slim side downward.

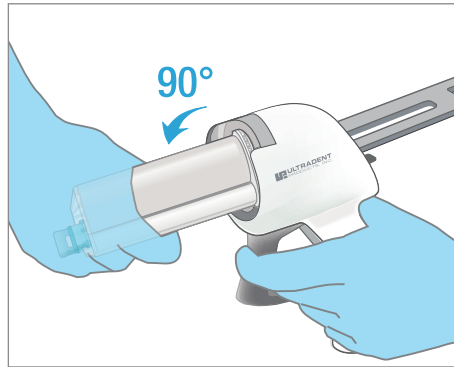


Lock the cartridge by turning it 90° counter-clockwise.

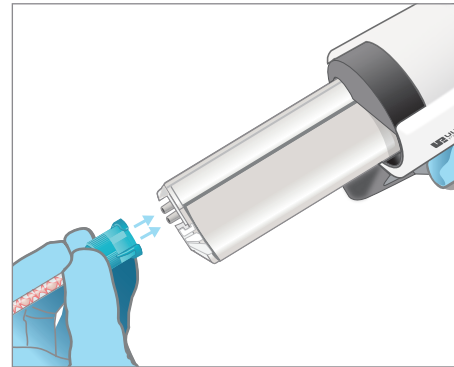


Attach the mixing tip. Note the corresponding notches.

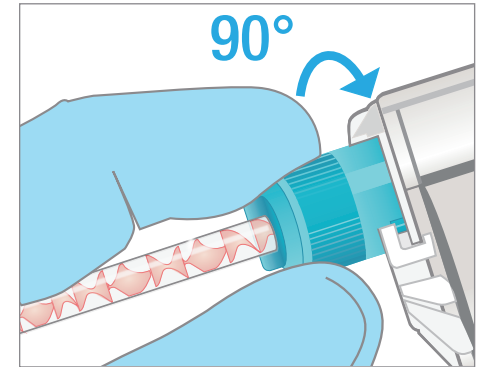

Turn the mixing tip 90° clockwise to lock it.

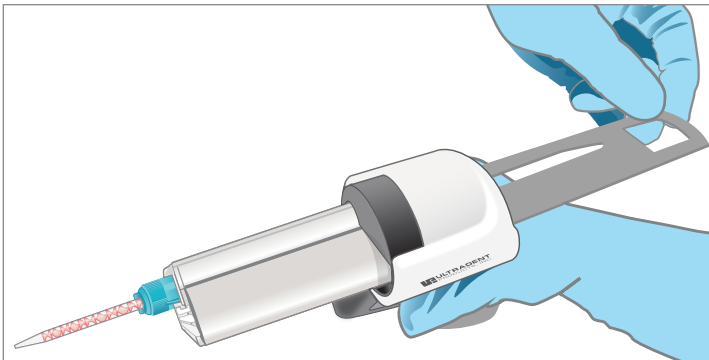

Push the plunger forward until it stops.

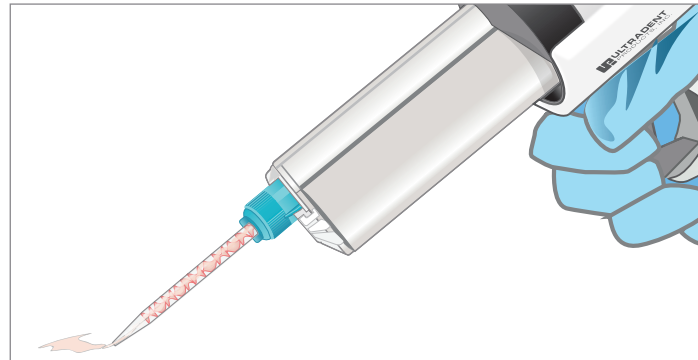

Pull the trigger until the material expresses out.